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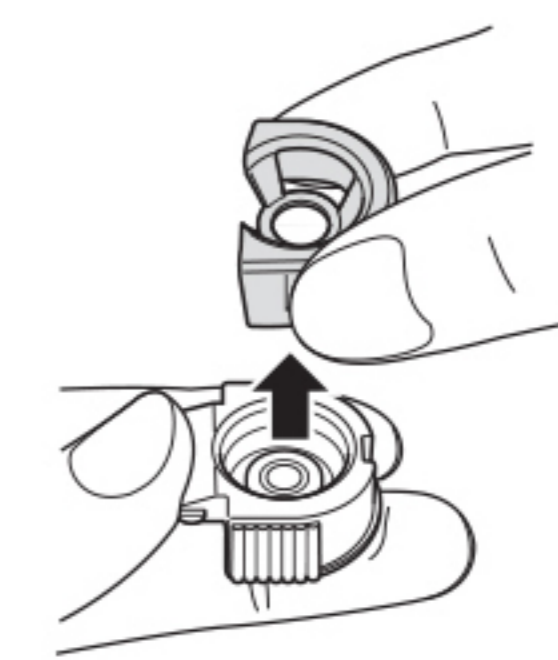
How to replace the mesh cap

The mesh cap is a consumable part.

Clement Clarke recommends to exchange the mesh cap after approximately 1 year. In case the unit nebulises poorly even after boiling in water (refer to “Disinfection by boiling”), replace the mesh cap. Remove the mesh cap and install a new one as described on page 10 (section “How to assemble the unit and fill the medication container”).

18

Troubleshooting

Trouble	Possible cause	Corrective action
The green power indicator turns on but the unit does not nebulise.	Medication or water has collected on the electrodes on the main unit.	Remove the medication or water. Refer to page 15.
	Excessive medication is collected on the mesh	Remove excessive medication. Refer to page 17.

If the unit does not nebulise properly after checking the possible causes above, contact your Clement Clarke dealer.

Replacement parts and accessories:

Meshcap
AC-adapter
Flexible adult mask
Flexible child mask
Mask adapter
Mouthpiece
Main unit cover

20

Troubleshooting

Trouble	Possible cause	Corrective action
The nebulisation rate is extremely low	Batteries are low (the orange battery low indicator flashes).	Replace the batteries with new ones. Refer to page 8.
	The mesh is stained or clogged.	1. Clean the mesh by boiling. Refer to page 15. 2. Replace the mesh cap with a new one. Refer to page 10.
The green power indicator does not turn on and the unit does not nebulise.	The batteries are low.	Replace the batteries with new ones. Refer to page 8.
	The capacity of rechargeable batteries is low.	Charge the batteries with a commercially available recharger.
	The batteries are inserted in the wrong direction.	Re-insert the batteries in the correct direction. Refer to page 8.
	The AC adapter is not correctly connected to the main unit.	Remove the AC adapter from the main unit and connect it correctly.
The green power indicator turns on but the unit does not nebulise.	The medication container is not installed correctly to the main unit.	Remove the medication container from the main unit and install correctly. Refer to page 11.
	The vibrator is not immersed in the medication.	Tilt the main unit in order to immerse the vibrator in the medication.
	The medication container is not filled with medication.	Fill with medication. Refer to page 11
	The mesh is broken.	Replace with a new one. Refer to page 10.

19

Specifications

This device complies with the provisions of the EC directive 93/42/EEC (Medical Device Directive).

Product name:	Electronic mesh nebuliser
Model name:	MEDIX MicroNeb
Power source:	3 VDC (two alkaline penlite batteries/Type AA/LR6) 2.4 VDC (two NiMH rechargeable batteries) AC adapter (optional) 230 VAC, 50 Hz
Power consumption:	Approx. 1.5 W
Ultrasonic frequency:	Approx. 180 KHz
Nebulisation rate:	Average 0.4 - 0.5 ml/minute*
Particle size (MMAD)**:	Approx. 5 µm*
Capacity of the medication container:	7 ml max.
Residual volume:	approx. 0.1 ml
External dimensions:	Approx. 38 (W) x 104 (H) x 51 (D) mm
Weight:	Approx. 97 gr (excl. batteries)
Battery life:	Approx. 8 days (30 minutes each day)
Operating conditions:	10° ~ 40°C, 30%~85% RH

Contents of the package: Main unit, main unit cover, medication container, mesh cap, mask adapter, mouthpiece, flexible child mask, flexible adult mask, 2 batteries (AA/LRG), instruction manual, warranty card, soft pouch

*Values measured with 0.9% NaCl solution and under temperature of 23°C; specifications may vary with medication used and environmental conditions.

**MMAD: Mass Median Aerodynamic Diameter.

MMAD is independantly measured at Bradford University, UK, Dr.John Dennis, according to the European nebuliser standard EN13544-1.

Documentation and/or specifications subject to change without prior notification.

 =class II

O/I =power off/on

 =Type B

 Read the instruction manual carefully

CE 0197

Clement Clarke International
Edinburgh Way
Harlow Essex
CM20 2TT

21